

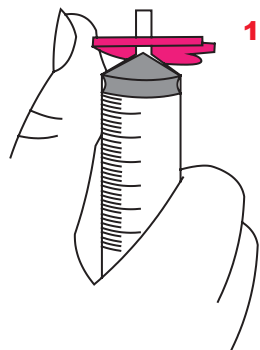


## Instructions for Use

### CheckCLIP FOR AMPOULE


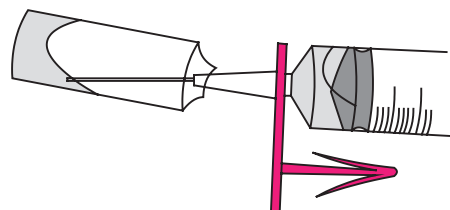


1. Face barb down and push CheckCLIP on to syringe using your thumb.

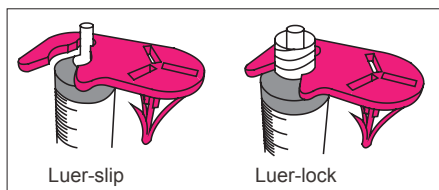


2. Snap open the ampoule and draw up the prescribed medication. Without the ampoule or syringe leaving your hands, proceed to step 3.

3. Push ampoule up on to CheckCLIP.

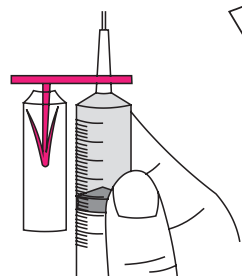
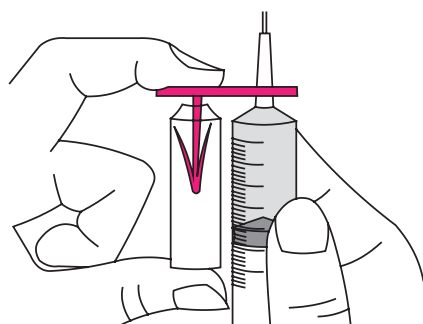


4. Double check before injecting.

### CheckCLIP FOR VIAL


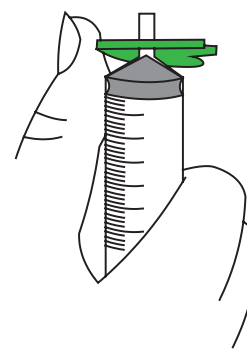


1. Face barb down and push CheckCLIP on to syringe using your thumb.

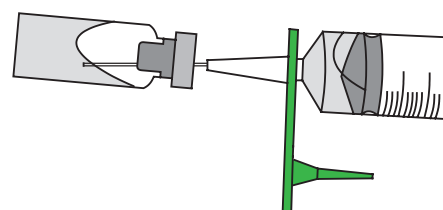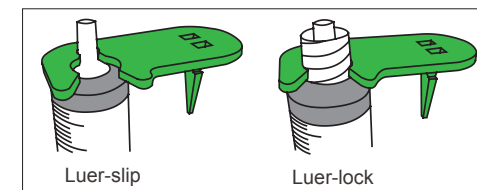

2. Draw up the prescribed medication and without the vial or syringe leaving your hands, proceed to step 3.

3. Using the needle track as a guide, push vial up on to CheckCLIP.

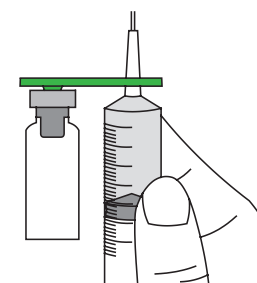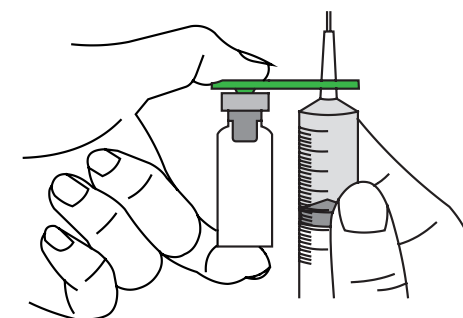

4. Double check before injecting.