


[BladeFLASK](#) / [Specifications](#) / [FAQ](#) / [Testimonials](#)

## 100 | Scalpel Blade Removal System

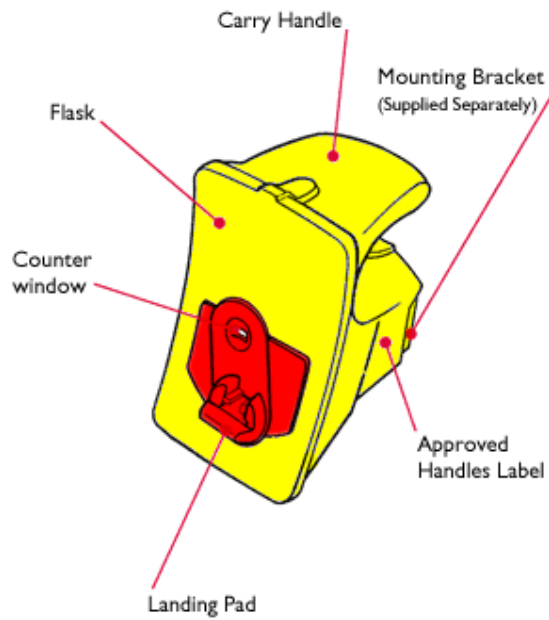

replacement.



## Removing blades using BladeFLASK

## The simply safer Qlicksmart method



Place blade flat onto landing pad - Sharp Edge Right / Tang "UP"

g Push scalpel handle into  
Blade<sub>FLASK</sub> parallel to landing  
pad - Until pronounced CLICK

Remove handle - Blade  
contained in  
Blade<sub>FLASK</sub>(removes up to  
100 bla